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## Baby Care Basics Nasal Aspirator

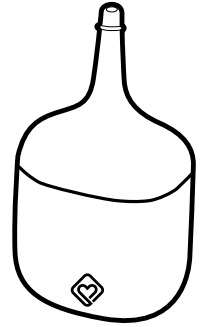
### IH688 User Guide

Read all instructions carefully before installing or using these items.

Keep these instructions for future reference.

#### BEFORE YOU BEGIN:

Remove all contents from packaging and discard box, and/or poly bags. Lay out and identify all parts. Do not return this product to the place of purchase. To address any issues or for helpful information on assembly/installation or use, please visit us at [www.safety1st.com](http://www.safety1st.com) to chat with us there. You can also text us at (812) 652-2525, email us at [support@djgusa.com](mailto:support@djgusa.com), or call Consumer Care at (800) 544-1108. Please provide the model number (IH688) and the manufacture date (located on package).

#### Nasal Aspirator

##### To Use:

1. Rinse prior to first use and dry thoroughly. Place your fingers and thumb around the aspirator bulb and squeeze any air out of the aspirator.
2. Hold your baby upright and gently place the tip of the aspirator into the baby's nostril so that it points partially up or down, but **NOT** directly towards the center of the nostril. Use the aspirator's insertion indicator ring as a guide; **DO NOT** push the aspirator into the nostril past this ring.
3. Slowly release pressure on the bulb and the suction will draw excess mucus from the baby's nasal passage.

**To Clean:** Aspirator is top-rack dishwasher safe or can be washed in warm, soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

FOR HOME USE ONLY. MADE WITHOUT BPA.

### ⚠ CAUTION

**NEVER** push the aspirator too far into the nostril and always wash and dry thoroughly after each use. **NEVER** use in baby's mouth or ears. When storing, keep away from heat and bright light.

This product is not a toy. **DO NOT** allow children to play with it.

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©2026 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com. Styles, colors, and labeling may vary. Les styles, les couleurs et l'étiquetage peuvent varier. Los estilos, colores y etiquetado pueden variar. Made in CHINA. Fabriqué en CHINA. Hecho en CHINA. Distributed by (distribué par/distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2233 Argentia Road, Suite 110, Mississauga, ON L5N 2X7. Representante exclusivo en Chile: DOREL JUVENILE CHILE S.A. Av. del Parque 4161, oficina 602 A, Huechuraba, Santiago, Chile - Fono: 56 2 24971500 - www.infanti.cl - www.dorel.cl. Importado y distribuido en Perú por: COMEXA COMERCIALIZADORA EXTRANJERA S.A. SUCURSAL DEL PERÚ - Los Libertadores 455, San Isidro, Lima 27, Lima, Perú RUC 20431062870 - Fono: 4413029 - www.infanti.com.pe. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Vía 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle, Benito Juárez C.P. 03100 México, Ciudad de México R.F.C. DJG140305GN9 Tel. (55) 67199202 03/26 4358-8843