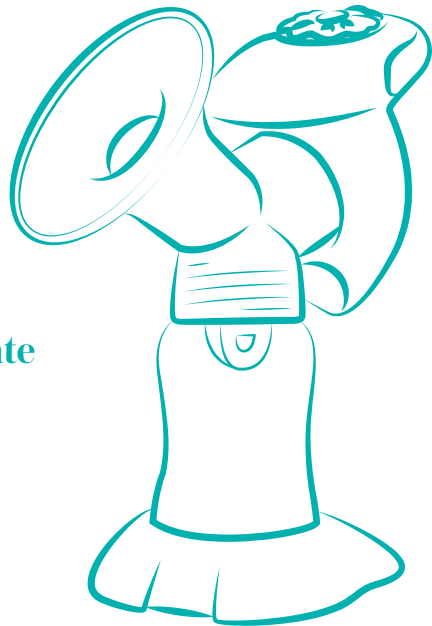
Advanced Single Electric Breast Pump

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For video instructions about how to use and care for your pump,
visit: www.evenflofeeding.com

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Thank you for choosing Evenflo Feeding to support you on your breastfeeding journey. If you have questions about the performance or operation of this pump, please contact our Parent's Ally Consumer Care Team.

For more information about breastfeeding or pumping, contact a lactation expert, your physician, or call us at USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at www.evenflofeeding.com.

Please find the most up-to-date instruction manual at www.evenflofeeding.com.

This pump is a single user medical device and should never be shared between mothers, resold, or given from one mother to another. Once the package has been opened, this product cannot be returned.

Please note that this pump should not be serviced or disassembled by the consumer. If you have any issues or concerns with the product, please contact our Parent's Ally Consumer Care Team.

Evenflo Feeding has established a robust support program for using your pump, including 1:1 pumping support by phone and video with the goal of making your pumping success our top priority.

After working with our Parent's Ally support team, our experts will determine what solutions we can offer, in accordance with our policies. For warranty information please visit our website at www.evenflofeeding.com/warranty.

Please Note: This pump is designed and intended for occasional use only, meaning no more than a few times a week, not more than once per day. Refer to the chart on the back of these instructions for examples of appropriate situations for occasional use. If this pump is used more often than recommended, your pump's life expectancy may diminish earlier, and it may negatively impact your milk supply, and/or pumping success. Moms who work full-time, or pump more than occasionally, should use the Evenflo Advanced Double Electric Breast Pump, which has been expertly developed for frequent pumping.

Important Safety Information.....	2-3
Identification of Parts.....	4
Before First Use.....	4-5
Assemble Your Breast Pump.....	6
Using Batteries.....	7
Using the AC Adapter.....	7
Check the Fit of the Flange.....	8
Using Your Breast Pump.....	9-11
After Every Use.....	11
Storing Your Breast Milk.....	11-12
Using Your Breast Milk.....	13
Maintenance.....	14
Troubleshooting.....	15-16

Read all instructions before using and save for future use.

When using this pump and other electrical products, basic safety precautions should always be maintained, including the following:

⚠️ DANGER: TO REDUCE THE RISK OF ELECTROCUTION

- **ALWAYS** unplug this product immediately after using.
- **DO NOT** use while bathing.
- **DO NOT** place or store this product where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.
- **DO NOT** reach for an electrical product that has fallen into water. Unplug the product immediately.
- **NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven.

⚠️ DANGER: TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- Use this product only for its intended use as described in these instructions.
- Breast pumps are personal care items, for one user only.
- This product should never be left unattended when plugged in.
- **NEVER** use while sleepy or drowsy.
- Close supervision is necessary when this product is used near children.
- **DO NOT** allow small children or pets to play with the pump unit, flange, AC adapter, etc. These parts may pose a choking or strangulation hazard.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- **DO NOT** use attachments not recommended by the manufacturer.
- **DO NOT** use this product if it is broken or any parts are missing.
- **NEVER** drop or insert any object into any opening of the pump unit.
- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening. See page 8 for proper flange fit.
- **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, dropped into water, or if any parts are broken or missing.
- Keep the cord away from heated surfaces.
- **DO NOT** use outdoors with the AC adapter or operate where aerosol spray products are being used, or oxygen is being administered.

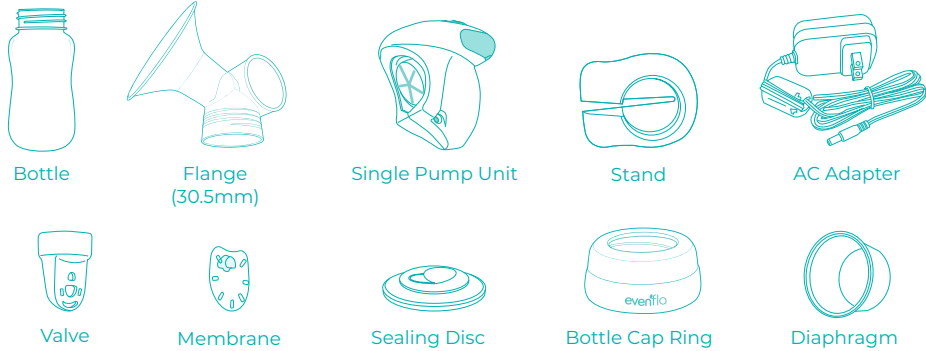
⚠️ DANGER: TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- Operate pump using ONLY 3 "AA" name-brand batteries (not included) or with the Evenflo AC adapter provided, and plug the adapter into a standard wall outlet.
- Operate the AC adapter on 100 – 240 volts 50/60 Hz AC only. Connect only to household outlet power supplies of this description. For use in countries outside North America, a plug converter (not included) may be required.
- Protect power cord and AC adapter cord. Route cords so that they are not walked on or pinched by items placed upon them.
- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump.
- Power adapter must be removed for battery operation. In case of a power outage, batteries will not act as a backup if the adapter is plugged into the pump.
- **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon-zinc), or rechargeable batteries; leakage, explosion, and/or fire could occur.
- Remove batteries before storing unit for a prolonged period of time.
- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds. See page 11 for proper use of breast milk.
- **DO NOT** re-freeze breast milk. After feeding, any unused breast milk left in the bottle can be used within 2 hours then discarded. See page 12 for proper milk storage.
- **ALWAYS** test temperature of liquid before feeding.
- Mothers should **NOT** breastfeed or feed expressed milk to their infants if the infant is diagnosed with galactosemia or the mother is diagnosed with HIV, HTLV-1/2, Ebola, has untreated brucellosis or tuberculosis, active HSV lesions on breast, or uses illicit street drugs.

Indications for Use: The Evenflo Advanced Single Electric Breast Pump is an electrically-powered suction device intended to express and collect milk from a lactating woman's breasts. This pump is intended for a single user.

SAVE THESE INSTRUCTIONS FOR FUTURE USE.

IDENTIFICATION OF PARTS


*28.0mm Flange Insert

* Additional sizes are available separately. See page 8.


BEFORE FIRST USE

IMPORTANT

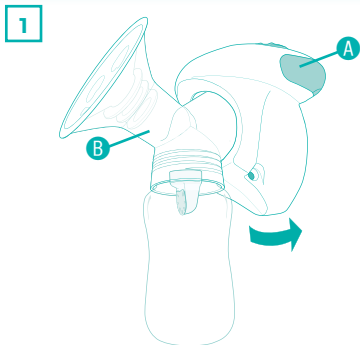Before using your breast pump, disassemble, clean and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.

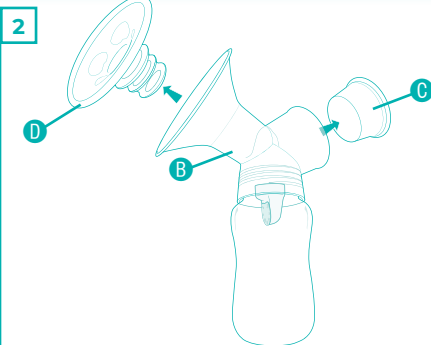Remove the pump unit **A** from flange assembly **B** by pulling up on pump unit, as shown.



Remove diaphragm **C** and flange insert **D** from flange assembly **B**.

BEFORE FIRST USE

3

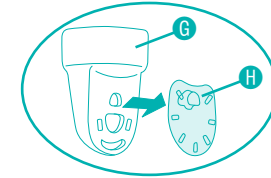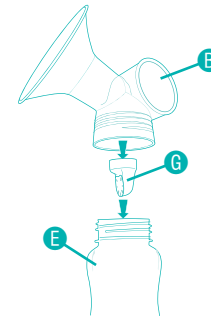Unscrew bottle **E** and twist valve **G** to remove from flange unit **B**. Remove membrane **H** from valve **G**.

Clean Your Pump Parts: To prevent warping, do not place anything on top of pump parts during sanitizing, washing, or drying. Do not assemble pump until parts are cool and dry.

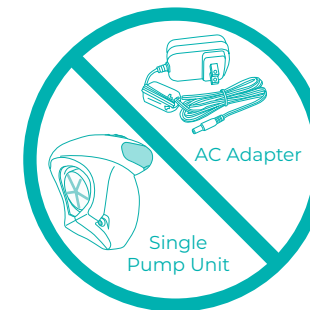
TO SANITIZE PARTS ON THE STOVETOP:

Place disassembled pump parts in a pan of boiling water for 5 minutes. Stir frequently. **DO NOT** allow parts to rest on bottom of pan as they may melt or become damaged.

TO SANITIZE PARTS IN THE MICROWAVE:

Use Evenflo Feeding Silicone Steam Sanitizing Bag according to instructions.

DO NOT BOIL OR WASH:


⚠ WARNING

NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur.


Before EVERY Use

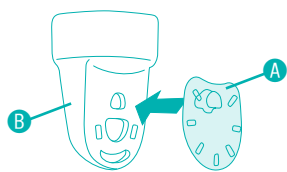- Wash your hands with soap and water before handling your pump.
- Ensure parts are clean and dry. Refer to cleaning instructions on page 11.
- Use care when handling the valve/membrane assembly to avoid damage.
- Visually inspect the individual components for cracks, chips, tears, or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.


1
Attach the membrane **A** to the valve **B**.

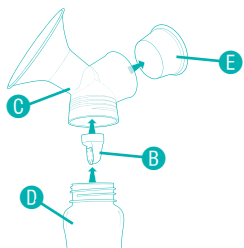Note: Make sure the membrane is attached and is lying flat against the valve, or the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this also will cause decreased suction and the pump to not work properly.

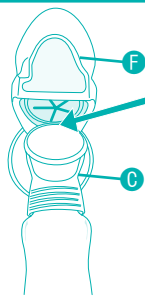
2
Push valve **B** onto the bottom of the flange **C**. Screw bottle **D** into bottom of flange. Install diaphragm **E** into flange assembly **C**.


3
Align the pump unit **F** to the flange assembly **C**, as shown.

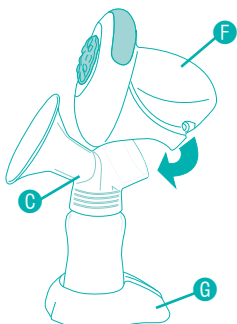Back View


Side View

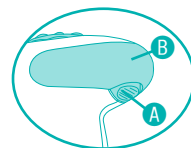In a downward motion, firmly snap the bottom of the pump unit **F** to the flange assembly **C**. You will feel a “click” when the two are secured tightly. Place the bottle into the stand **G**, as shown.

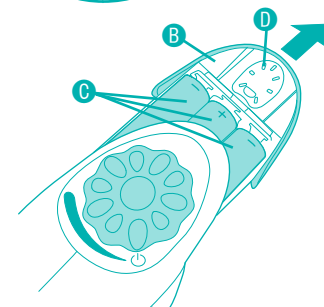If you plan to use batteries, push battery drawer release button **A**, slide battery drawer **B** out and load three alkaline “AA” batteries **C** (not included), as shown. Slide the drawer **B** closed. For best performance, brand name, non-rechargeable batteries are recommended. Generic batteries may impact pump performance.

Note: You may store a spare membrane **D** in battery drawer, as shown.

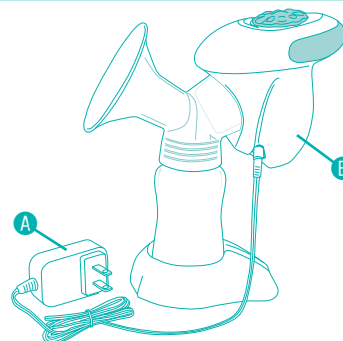
Note: Power adapter must be removed for battery operation. In case of a power outage, batteries will not act as a backup if the adapter is plugged into the pump.

⚠ WARNING

- **DO NOT** mix old and new batteries.
- **DO NOT** mix battery types; leakage, explosion, and/or fire could occur.
- Remove batteries before putting product into storage.

USING THE AC ADAPTER

To use the AC adapter, **A**, remove batteries from pump unit.

Plug AC adapter into the pump unit **B**, then plug the adapter into 100 – 240V 50/60 Hz AC current only (standard household outlet). For use in countries outside of North America, a plug converter (not included) may be required.

Unplug the adapter after each use.

Note: AC adapters are not interchangeable between Evenflo Models

⚠ WARNING

- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump. If you lose the AC adapter contact Parent’s Ally Consumer Care Team.
- **ALWAYS** remove plug from product and wall by pulling on the plug/housing.
- **DO NOT** pull on the cord, as this may damage the cord.
- **NEVER** operate this product if it has a damaged cord or plug.

CHECK THE FIT OF THE FLANGE

The Advanced Single Electric Pump works with the Evenflo AdvancedFit™ Flange System. The Advanced Single Electric Pump comes with the Large 30.5mm base flange size and the Medium 28.0mm silicone flange insert. Additional sizes are available separately for purchase, allowing you to find your best fit.

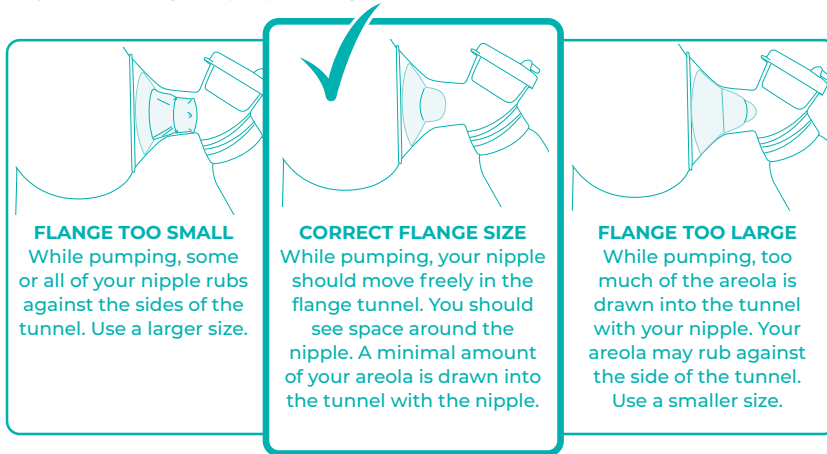
THE ADVANCEDFIT™ FLANGE SYSTEM AVAILABLE SIZES INCLUDE:

- 21.0mm (insert)
- 24.5mm (insert)
- 28.0mm (insert)
- 30.5mm (no insert)
- 33.0mm (insert)
- 36.0mm (no insert)

To use silicone flange insert, press it into the tunnel of the hard flange firmly. Then ensure that the outside edges of the insert wrap around the hard flange.

Correct Flange Size vs. Too Small or Too Large

A proper flange size is needed for comfort and to maintain your milk supply. Your flange size can change several times throughout your journey. Recheck your flange fit frequently to ensure you are using the proper flange size.

FLANGE TOO SMALL

While pumping, some or all of your nipple rubs against the sides of the tunnel. Use a larger size.

CORRECT FLANGE SIZE

While pumping, your nipple should move freely in the flange tunnel. You should see space around the nipple. A minimal amount of your areola is drawn into the tunnel with the nipple.

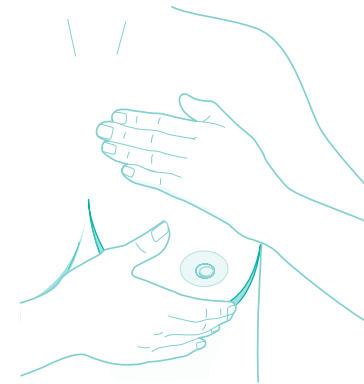
FLANGE TOO LARGE

While pumping, too much of the areola is drawn into the tunnel with your nipple. Your areola may rub against the side of the tunnel. Use a smaller size.

IMPORTANT

- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort.
- Included flange sizes may not fit all breast and nipple sizes, but additional sizes can be purchased separately. If you are having issues after trying all included sizes, please contact our Parent's Ally Consumer Care for assistance.

USING YOUR BREAST PUMP


1

Stimulate your milk release before attaching the pump to maximize your pumping results by:


- Relaxing and thinking about your baby.
- Gently massaging your breasts, starting on the outside of your breast working down toward the nipple.
- Applying warm, moist compresses; continue until you can hand-express a few drops of milk.

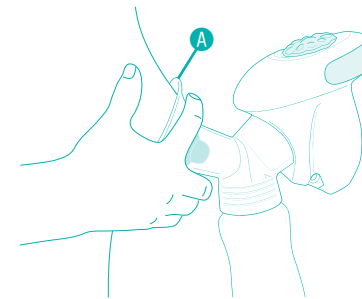Now you should be ready to use your pump.


2

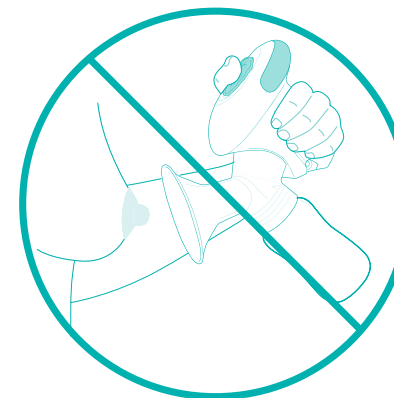For best results, hold the pump by the flange **A**. Place the flange **A** against your breast with your nipple centered in the flange tunnel. Press directly against your breast to create a good seal.


DO NOT hold the pump unit while pumping. For best results, hold the flange directly.

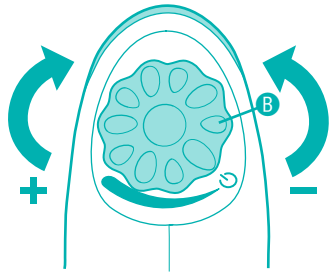Note: Be sure the outside edge of the flange has a complete seal against the breast without excessive pressure.

3

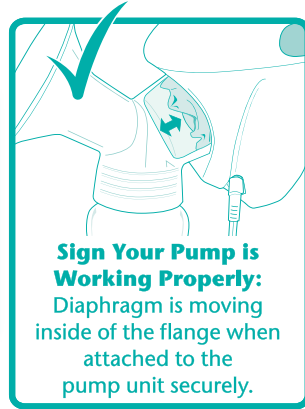
Turn on the pump by turning the control dial **B** clockwise to begin pumping. To increase suction, gradually turn the control dial clockwise to reach the level that is most comfortable for you. To reduce suction, turn the control dial **B** counterclockwise.

Sign Your Pump is Working Properly: Diaphragm is moving inside of the flange when attached to the pump unit securely.


IMPORTANT

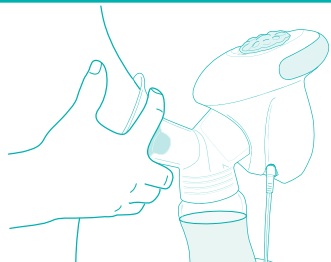If you feel pain in your breast or nipple, turn the pump off (see next page) or slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort at every suction level, turn the pump off and contact your healthcare provider for assistance.

4

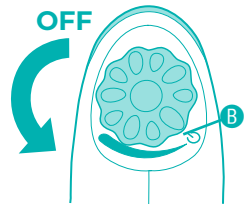Continue pumping until milk is no longer flowing. Do not fill bottle above the 5 oz/150 mL line.


5

After milk has been expressed from one breast, turn control dial **B** counterclockwise until pump unit is in the OFF position. If necessary, slide a finger between the flange and your breast to break any remaining suction, then remove the flange.

Massage your other breast to stimulate a milk release, as before, and repeat steps 2 through 4. If desired, massage the first breast again and repeat on both sides to maximize expression.

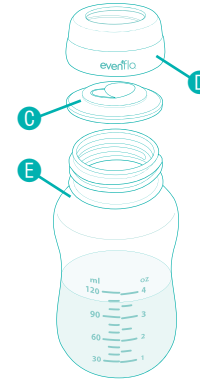6

Carefully unscrew bottle from pump. Place the sealing disc **C** into the cap ring **D** and screw securely onto the bottle **E**.

Your breast milk is now ready for storage.

Note: Information on storing breast milk can be found on page 11.

AFTER EVERY USE

WARNING

NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur.

To ensure that your breast pump performs well, follow these steps after every use:

- Completely disassemble the flange assembly, see page 4.
- Wipe the pump unit with a clean, damp cloth.
- Rinse the flange assembly (all parts EXCEPT the pump unit and AC adapter) in cool water to remove milk residue.
- Hand wash with warm, soapy water (approx +30°C / +86°F).
- To prevent warping, do not place anything on top of pump parts during sanitizing, washing, or drying. Do not assemble pump until parts are cool and dry.
- Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher (take care not to rest dishes on top of pump parts).
- Use care when handling the valve/membrane assembly during cleaning to avoid damage. Visually inspect the individual components for cracks, chips, tears, or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.

STORING YOUR BREAST MILK

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart on the following page as a guide for storing your breast milk for future use.

Note: In hot weather, chill milk immediately to prevent spoilage.

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart below as a guide for storing your breast milk for future use.

Note: In hot weather, chill milk immediately to prevent spoilage.

Breast Milk Storage Guidelines*

	Room Temperature	Refrigerator	Freezer
Freshly Expressed Breast Milk	Up to 4 hours	Up to 4 days	Within 6 months is best Up to 12 months is acceptable
Thawed Breast Milk (frozen)	1 - 2 hours	24 hours	NEVER re-freeze milk after it has thawed

* CDC. *Proper Storage and Preparation of Breast Milk, Revised 2020.* https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm

* ABM Clinical Protocol #8: *Human Milk Storage Information for Home Use for Full-Term Infants, Revised 2017.* <https://abm.memberclicks.net/assets/DOCUMENTS/PROTOCOLS/8-human-milk-storage-protocol-english.pdf>

Note: Breast milk should be stored in the refrigerator or freezer compartment away from the door to avoid changes in temperature.

For models that include a cooler bag:

1. Place frozen ice packs in the included cooler bag.
2. Place the sealed bottle(s) inside and zip the bag closed.
3. When frozen ice packs are used and the cooler bag is kept at room temperature of 68 – 77° F (20 – 25° C), breast milk can be safely stored for up to 10 hours.

Note: Avoid storing the cooler bag in a hot closet, cabinet, or car.

1. Thaw frozen milk slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, you may warm refrigerated milk:
 - Hold the sealed bottle under warm, running water, or
 - Use a bottle warmer to the manufacturer's instructions, or
 - Heat water in a cup or other small container, then place sealed bottle in the water to warm.
 - **DO NOT** heat breast milk for more than 20 minutes.
3. The milk fat may separate while stored. Briefly shake or swirl milk to mix before offering it to baby.
4. If feeding from the collection bottle, you must use **Evenflo Balance + Standard Nipples**. Other brands or types of nipples will not work effectively. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the **TOP** of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care not to over or under tighten. *Evenflo Balance + Standard Nipples sold separately.*
5. Always test the temperature of the milk before feeding.

IMPORTANT

- **NEVER** microwave human milk or heat directly on the stove.
- If baby does not finish breast milk at one feeding, unfinished amount must be discarded within 2 hours.
- Cool freshly pumped breast milk before adding to refrigerated or frozen breast milk.

For optimal performance, keep spare parts on hand. Replace at the first sign of wear or if you pump less milk than expected. It is normal for pump parts to lose elasticity and wear over time with use, washing, and sanitizing. Here is a guide for pump maintenance.

Breast Pump Part Replacement Guide

Part	Pumping for occasional use (see PAGE 1)
 Membrane	2-3 months
 Diaphragm	3-6 months
 Valve, and flange (hard, non-moving pieces)	6 months

Problem	Possible Cause	Action
Pump won't turn on	No batteries or low batteries	Make sure fresh, brand-name batteries are installed correctly (PAGE 7).
	AC adapter cord is plugged into pump and batteries are inserted	If using batteries, adapter cord must be removed from pump for batteries to work.
	AC adapter is not plugged into a functioning wall outlet	Plug AC adapter into known functioning wall outlet. Test outlet with other electrical devices.
		Try operating pump with fresh, brand-name batteries.
	Batteries installed when trying to use AC adapter	Remove batteries before using AC adapter.
Milk won't flow into the bottle	The membrane is stuck to the valve	Clean the valve and reassemble, per step 1 in Assembling Your Breast Pump section (PAGE 6).
	Not stimulating milk release before expressing	Stimulate your milk release with massage and hand expression before placing pump to breast (PAGE 9).
	Fast milk flow	Turn the control dial counterclockwise for lower suction (PAGE 10).
Little to no suction	Motor not assembled to flange properly	Firmly snap motor onto flange (PAGE 6).
		Once motor is firmly snapped into place, hold pump by flange (PAGE 6).
	Diaphragm not inserted into flange	Properly install diaphragm. (STEP 2 PAGE 6)
	Diaphragm pinched	Disconnect motor, reseal diaphragm, and reattach motor (PAGE 6)
	Diaphragm is damaged or worn	Check diaphragm for signs of wear, holes, cracks, etc. Replace damaged components.
	Valve not assembled to flange	Attach the valve firmly to flange (PAGE 6).
	Membrane worn, torn or not assembled to valve	Replace membrane or ensure that it is connected properly to the valve (PAGE 6).
Dried milk in pump	Follow cleaning instructions (PAGE 11). Pay extra attention to the valve and membrane.	

Continued on next page...

Problem	Possible Cause	Action
Little to no suction, cont.	Membrane not sealing	Make sure valve and membrane are clean, with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve (PAGE 6) .
	Flange insert not fully seated into flange	Properly install flange insert (PAGE 8) .
	Poor seal between breast and flange	Reposition the flange to get a better seal. Make sure flange seals directly onto breast with no clothing between breast and flange. (PAGE 9) .
	Suction setting too low	Turn the control dial clockwise for stronger suction (PAGE 10) .
	No batteries or low batteries	Make sure fresh, brand-name batteries are installed correctly (PAGE 7) .
High suction	Suction settings too high	Turn the control dial counterclockwise for lower suction (PAGE 10) .
	Damaged diaphragm	Inspect diaphragm before each use, replace damaged components.
Discomfort	Damaged diaphragm	Inspect diaphragm before each use, replace damaged components.
	Suction settings too high	Turn the control dial counterclockwise for lower suction (PAGE 10) .
	Using incorrect flange size	Using a flange that is too small or too large can cause discomfort. See Flange Sizing (PAGE 8) .
Expressing little to no milk when using the pump	Not stimulating milk release before expressing	Stimulate your milk release with massage and hand expression before placing pump to breast (PAGE 9) .
	Missing parts	Ensure diaphragm, valve, and membrane are properly assembled
	Pumping after breastfeeding	Wait 45-60 minutes after breastfeeding before pumping.
	Not using pump as intended	This is an occasional use pump, not intended for daily use (SEE INSIDE FRONT COVER) .

For warranty information please contact us at USA/Canada: 1-855-EFI-BABY (334-2229), by mail at Evenflo Feeding Inc., Attn. Consumer Resources, 9277 Centre Pointe Drive, Suite 160, West Chester, OH 45069, or visit our website at www.evenflofeeding.com/warranty.

Still having problems? Our Parent's Ally Consumer Care Team can help.

Our Parent's Ally Consumer Care Team will work with you to determine what solutions we can offer, in accordance with our policies.

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