M+D X-FOLD™

Shower Chair

Chaise De Douche

Silla De Ducha

Instructions For Use
Instructions D'utilisation
Instrucciones De Uso

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

Where Real Life Inspires Innovation


Designed with insight from real lives, elevated by next-generation engineering and innovative materials — because everyday support should feel extraordinary.

- *The M+D by Drive® Team*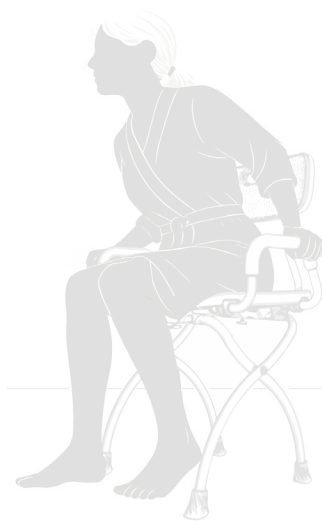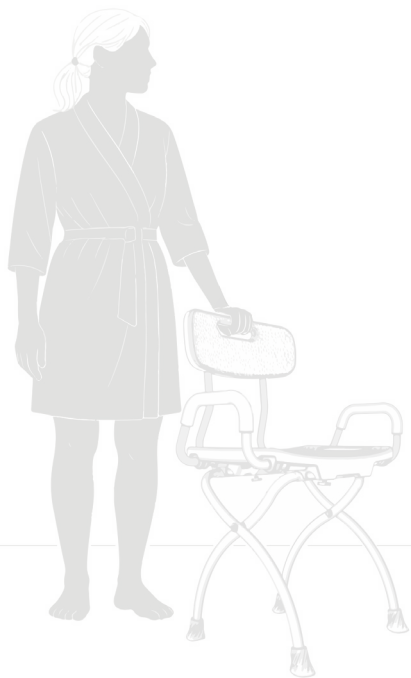

Parts List

- 1 Backrest
- 2 Backrest Support Tube
- 3 Left Arm Rest
- 4 Right Arm Rest
- 5 Seat
- 6 X-Fold Leg Frame
- 7 Foot
- 8 Thumb Screw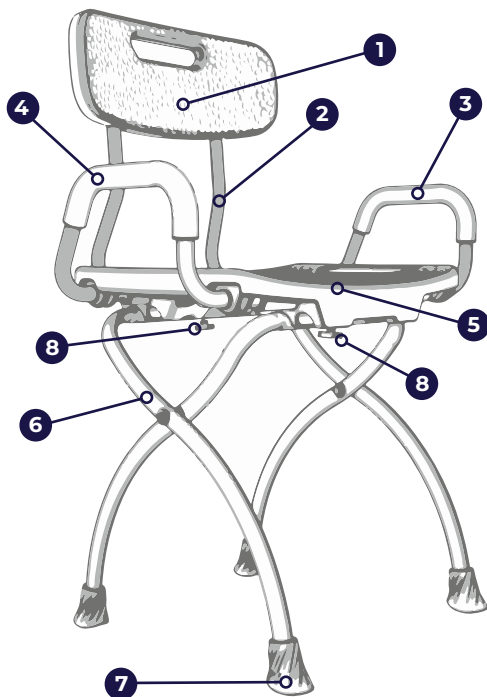

WEIGHT CAPACITY: 300 LB / 136 KG

SEAT HEIGHT RANGE: 20.1" - 23.6" / 510mm - 600mm


Assembly Instructions


Tutorial Videos

Scan QR code for link to tutorial videos

- 1 Unbox and lay out all components.
- 2 Lay shower chair seat upside down on a flat surface.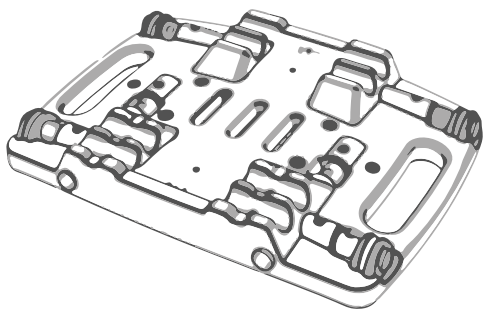



- 3 To attach the legs, insert the X-Fold leg frame to achieve the desired height. The leg frame should snap into place; you will hear and feel a click.
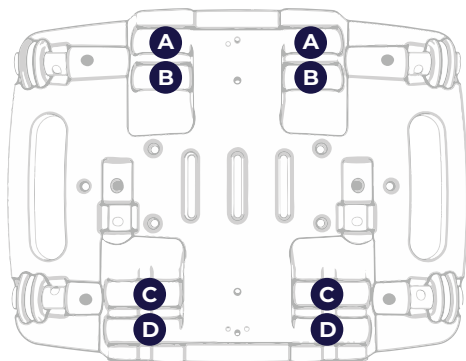


Assembly Instructions



Low Setting – Insert leg frame into Section A and Section D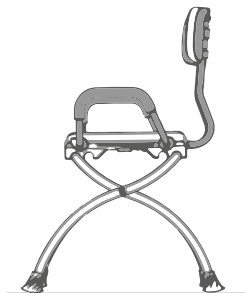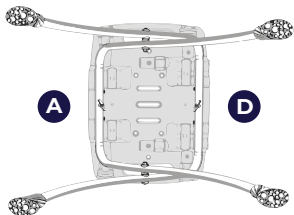

Medium Setting – Insert leg frame into Section B and Section D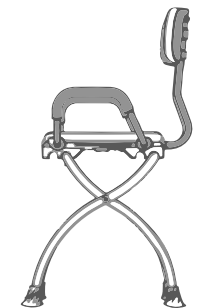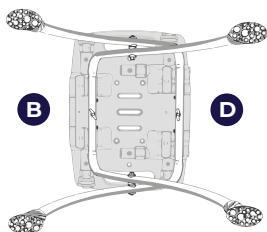

High Setting – Insert leg frame into Section B and Section C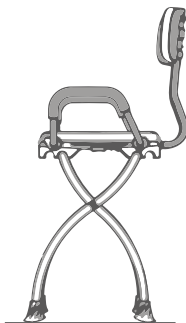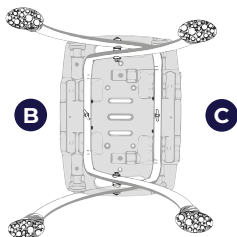


Assembly Instructions


4 Secure leg tubes to seat bottom.

- Use (2) thumb screws with curved washer, insert screw into center holes on both front and rear leg tube holes.
- Tighten each thumb screw until finger tight to secure the X-Fold leg frame onto the seat.

Note: To keep from damaging thumb screw threads, make sure to align thread of thumb screw with threaded insert in seat bottom.

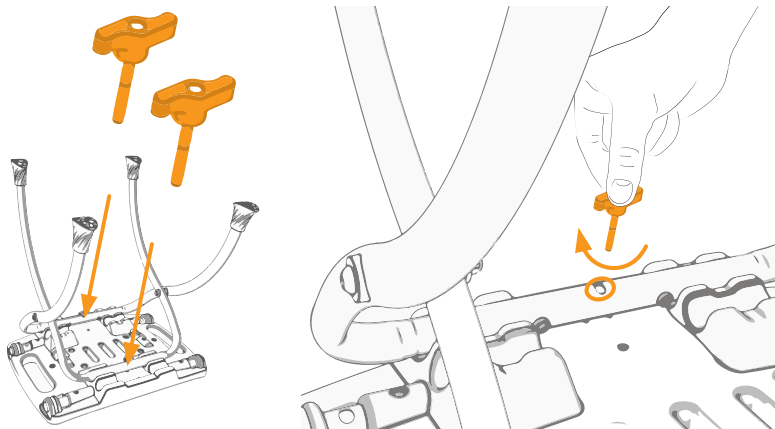
5 Assemble backrest by locating the curved shaped backrest support tubes and backrest.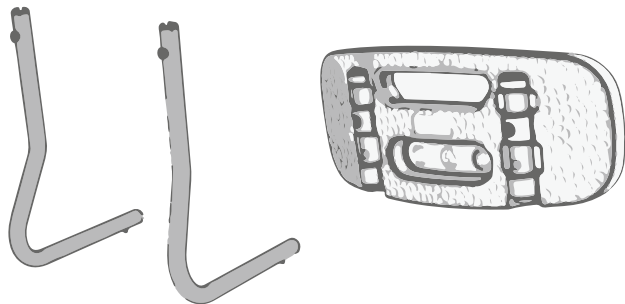


Assembly Instructions

- 6** Use your thumb to depress snap button on support tube and slide it into the backrest opening.

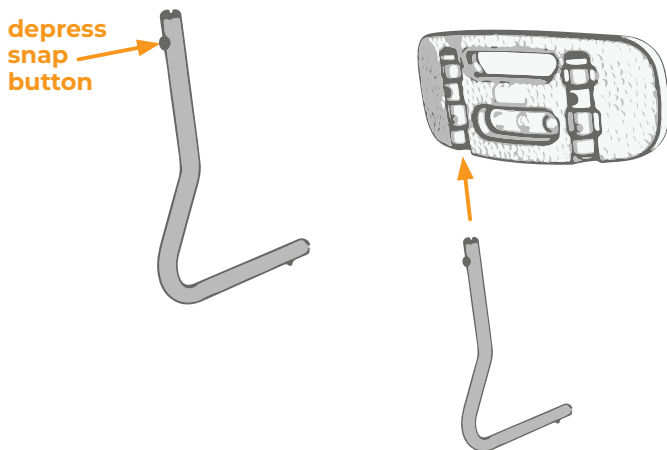
- 7** Continue sliding until the snap button fully protrudes through the adjustment hole, you will hear or feel a click when the sections are securely locked together.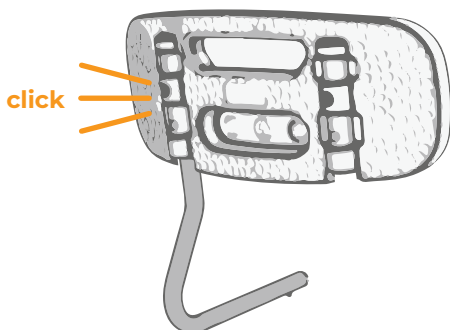


Assembly Instructions

- 8 Repeat on other side.

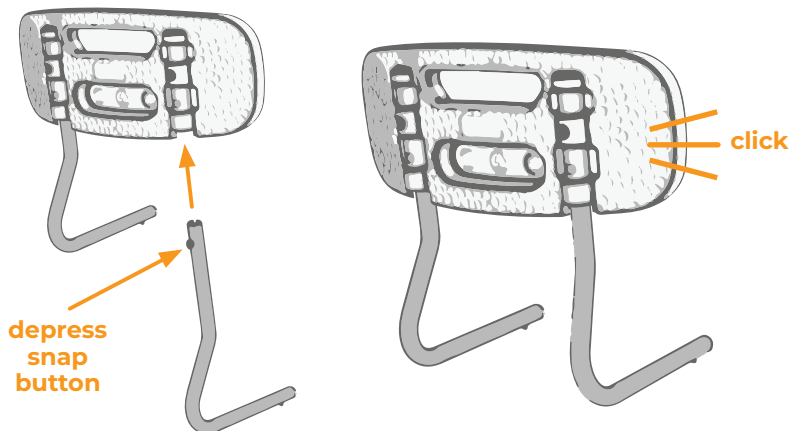
- 9 To attach the backrest to the seat, align the support tubes into the seat tube openings.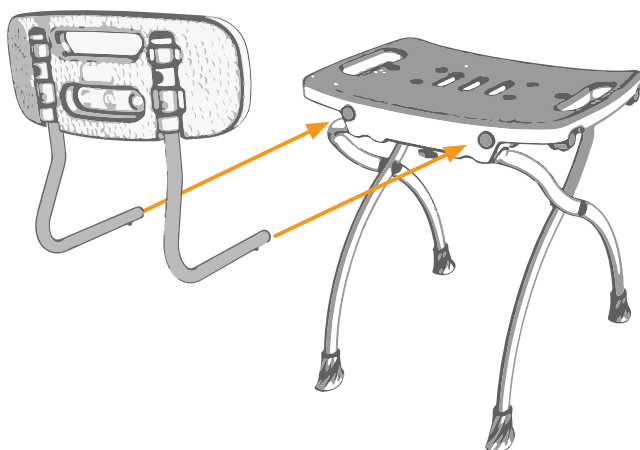


Assembly Instructions

- 10** Use your thumbs to depress both snap buttons on each support tube and slide it into the seat tube openings.

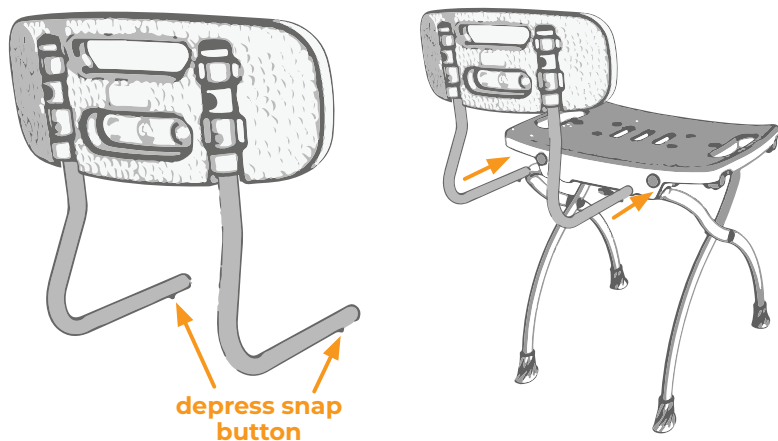
- 11** Continue sliding until the snap buttons fully protrude through the adjustment holes; you will hear or feel a click when the sections are securely locked together.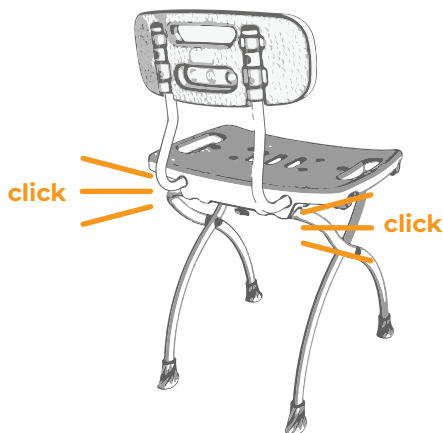


Assembly Instructions

- 12** To assemble the arm rests, align arm rest tube ends into the side seat tube openings.
- Note:** The angled side of the arm rest faces backrest. The 90-degree side is positioned in seat hole furthest from backrest.

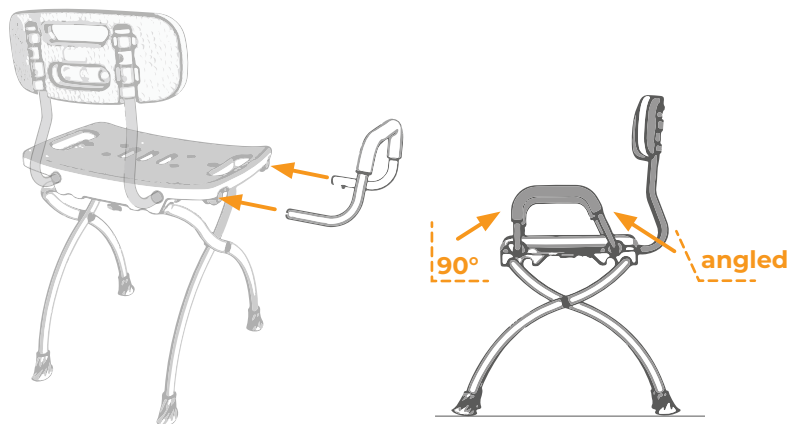
- 13** Use your thumbs to depress both snap buttons on the arm rest and slide it into the seat tube openings.
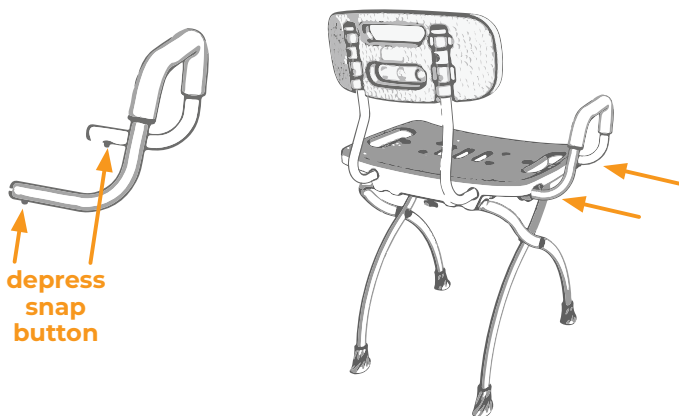


Assembly Instructions

- 14** Continue sliding until the snap buttons fully protrude through the adjustment holes; you will hear or feel a click when the sections are securely locked together.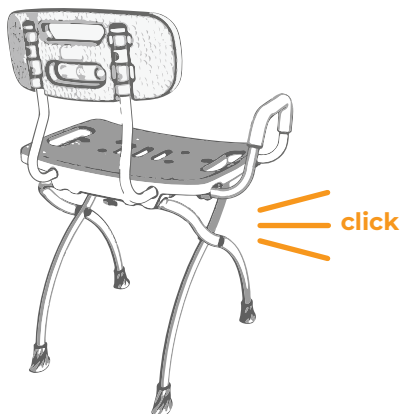

- 15** Repeat on the other side.

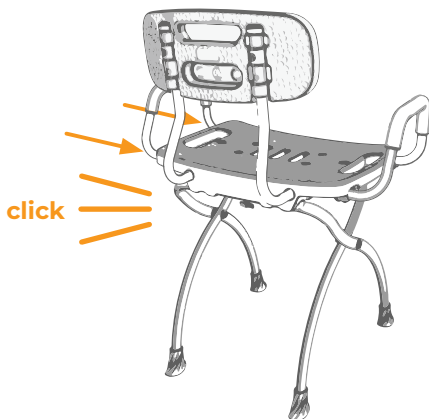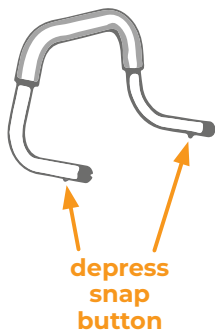Safety Check

- Check the natural rubber feet on the X-Fold leg frame for rips, or other damage, and ensure that each leg has a rubber tip. Replace any tips that are damaged or missing.
- All four feet **MUST** touch the floor at all times.
- **DO NOT** stand on the chair.

Maintenance and Cleaning

Clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation.

If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, rubber feet, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

Patents: www.drivemedical.com/patents

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All rights reserved.
Mobility Designed® is a trademark of Medical Depot, Inc.
© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.