Advanced Milk Collection Bottles

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INSTRUCTIONS:

To Clean: Before first use, boil all parts for 3 minutes to sanitize and cool to room temperature (at least 5 minutes). After each use, wash all parts thoroughly in warm, soapy water. Rinse and dry thoroughly. Dishwasher safe (top rack only). May be cleaned in a steam sanitization bag.

To Prepare Milk: Before feeding baby, hold the sealed bottle under warm running water until milk is at room temperature. Gently swirl the sealed bottle several times to mix the milk fat that has risen to the top during storage. Always test temperature of liquid before feeding baby.

⚠️WARNING: DO NOT HEAT IN MICROWAVE OVEN. Microwaving can destroy important nutrients in breast milk and can heat liquids quickly and unevenly. Burns or scalding may result. Breast milk should not be refrozen once it has been thawed. Do not pour freshly pumped breast milk onto frozen breast milk because thawing can occur.

To Store: Pump directly into storage bottle or pour expressed milk into storage bottles. If freezing milk, fill only 3/4 full, as milk expands with freezing. Ensure bottle is sealed by pushing round silicone disc through top of cap ring, until it is seated snugly to the cap ring. Screw cap ring onto bottle tightly. Write date and time milk was expressed on bottle using a grease pencil or write on tape. Use storage chart as a guideline:

| Storage Location | Temperature | Length of Time Recommended* |
|-------------------------------------|-------------|-----------------------------|
| room temperature | 73-77°F | up to 4 hrs. |
| refrigerator | 32-39°F | up to 5-7 days |
| self contained refrigerator freezer | 0°F | up to 3-4 mos. |
| deep freezer | 0°F | up to 6-12 mos. |

Use thawed milk within 24 hours. Never refreeze thawed milk.

*Human Milk Banking Association of North America. Raleigh, NC: 3rd Edition, 2011

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To Use with Balance + Standard Neck Nipple: Before first use, boil all parts for 3 minutes to sanitize and cool to room temperature (at least 5 minutes). After each use, wash all parts thoroughly in warm, soapy water. Rinse and dry thoroughly. Dishwasher safe (top rack only). May be cleaned in a steam sanitization bag.

Squeeze nipple base and insert through the TOP of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand tighten cap ring with care to not over or under tighten. Prior to feeding, pinch the Proflo™ valve between two fingers to separate the sides to ensure proper venting.

For best feeding results, ensure baby's lips are flared around the nipple base without being overstretched during use.

WARNING: Evenflo Balance + Nipples are designed for use with Balance + Bottles only. The use of other Evenflo or competitor nipples is not advised as it could cause this bottle system to not function properly.

To prevent injury, adult must be present during use.

Do not attempt to enlarge hole as this will cause tearing. Any tear in the nipple can lead to the separation of a piece, which could be a choking hazard. Inspect frequently and stop use if you see a tear or crack.

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©2019 EVENFLO FEEDING, INC.