DERMATIX® SILICONE SHEET

with Silon® technology
Scar treatment

AVAILABLE IN TWO FORMS*:

DERMATIX® SILICONE SHEET CLEAR

Virtually invisible! Apply on exposed areas.

Perfect for face & hands – can even apply make-up over it.

DERMATIX® SILICONE SHEET FABRIC

With a silky fabric backing for optimal comfort and convenience under clothing.

DESCRIPTION:

Dermatix® Silicone Sheet [Clear and Fabric] are breathable and washable adhesive sheets made using advanced novel silicones.

USE:

Consistent use of Dermatix® Silicone Sheet [Clear or Fabric] can reduce existing prominent immature hypertrophic scars and keloids or prevent similar newly forming scars whilst helping to reduce their pain, itchiness, redness or height.

Product usage for at least 2 months and on average 3-6 months is normally sufficient to achieve maximum effect on the scar. However, individual results may vary since factors such as the type or age of the scar, its severity, patient's age and genetics will have an effect on the treatment duration of the scar.

INSTRUCTIONS:

STEP 1: Remove the product from its package. If necessary, use scissors to trim the product to size. It should extend slightly beyond the area of the scar.

STEP 2: Peel away the protective liner and save it for later use. Place the product "sticky side" down, directly on the scarred area. Keep the product in place for at least 12 hours and up to 23 hours per day (to allow sufficient time for step 3).

STEP 3: At least once a day, wash the scarred area and both sides of the product with mild soap and warm water. Do not use a soap that contains moisturizers, lotions, or oils. A plain basic soap is recommended.

Rinse the product thoroughly to remove all traces of soap.

Allow the product to dry before reapplying. Between uses, store the product on a clean dry surface such as the protective liner. Eventually, the product will lose its ability to remain sticky after repeated uses, but it normally lasts approximately 3 to 6 weeks. Proper care of the product is critical in order to maintain stickiness for ease of use.

SPECIAL RECOMMENDATIONS AND PRECAUTIONS:

WARNING! DO NOT USE ON OPEN WOUNDS.

For external use only.

Do not use creams, lotions or ointments with this product as they may interfere with the performance of the product and/or cause skin irritation.

In case of serious incidents, please immediately contact Bio Med Sciences, Inc. and your National Competent Authority.

CAUTION:

As with any topical material applied directly to the skin's surface, some type of reaction (usually a sweat rash) is possible. Stop using the product if you have a reaction. After your skin has returned to normal, you may reapply the product for short periods and gradually increase the hours of usage. Monitor the scarred area for signs of reaction. Continue to increase the wearing time until a comfortable maximum has been reached. If you continue to have a problem, stop use and consult your physician.

Silon® is a registered trademark of Bio Med Sciences, Inc.

© All rights reserved. This product is subject to US & Foreign Patents including US #'s 5,656,279, 5,980,923 & 7,087,135. Form ID: Dermatix IFU-EU REV 07/2023

*Not all countries supply all presentations.

Bio Med Sciences, Inc., 999 Postal Road, Allentown,
PA 18109, USA

Emergo Europe, Prinsessegracht 20 - 2514 AP The Hague,
The Netherlands

Mylan Ireland Limited, Northern Cross, Dublin 17, Ireland

UK: Viatris UK Healthcare Ltd., 20 Station Close, Potters Bar, England, EN6 1TL
Ireland: Mylan IRE Healthcare Limited, Unit 35/36,
Grange Parade, Baldoyle Industrial Estate, Dublin 13

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© All rights reserved. This product is subject to US & Foreign Patents including US #s 5,656,279, 5,980,923 & 7,087,135. Form ID: Dermatix IFU-EU REV 07/2023
© All rights reserved. This product is subject to US & Foreign Patents including US #’s 5,656,279, 5,980,923 & 7,087,135. Form ID: Dermatix IFU-EU REV 07/2023